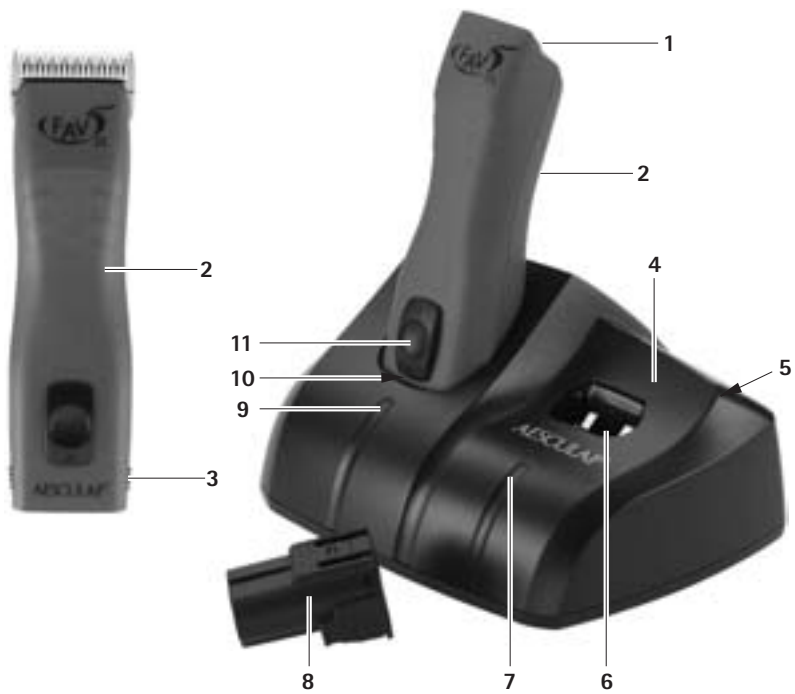
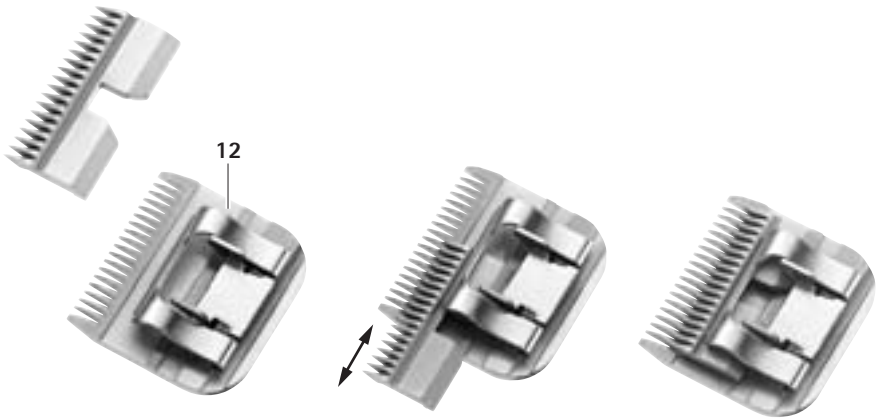
- (GB)** **Instructions for use/Technical description**
Battery-powered clipping machine FAV5 CL
- (USA)** **Battery-powered clipping machine FAV5 CL**
- (D)** **Gebrauchsanweisung/Technische Beschreibung**
Akku-Schermaschine FAV5 CL
- (F)** **Mode d'emploi/Description technique**
Tondeuse à accu FAV5 CL
- (E)** **Instrucciones de manejo/Descripción técnica**
Esquiladora a batería FAV5 CL
- (I)** **Istruzioni per l'uso/Descrizione tecnica**
Tosatrice ad accumulatore FAV5 CL
- (NL)** **Gebruiksaanwijzing/Technische beschrijving**
Accu-scheermachine FAV5 CL
- (RUS)** **Инструкция по применению/Техническое описание**
Машинка для стрижки, с аккумулятором, FAV5 CL

Battery-powered clipping machine FAV5 CL

Legend

- 1 Blade adapter
- 2 Battery-powered clipping machine FAV5 CL
- 3 Battery release lock
- 4 Charger
- 5 Mains power cord (with device plug)
- 6 Charging bay (for Li-ion battery)
- 7 Charge state indicator (for Li-ion battery)
- 8 Li-ion battery
- 9 Charge state indicator (for battery-powered clipping machine FAV5 CL)
- 10 Charging bay (for battery-powered clipping machine FAV5 CL)
- 11 On/Off switch
- 12 Blade

Symbols on product

	<p>Adhere to instructions for use</p>
	<p>Marking of electric and electronic devices according to directive 2002/96/EC (WEEE), see Chapter Disposal</p>

Contents

- 1. Safe handling 3
- 2. Product description 3
 - 2.1 Scope of delivery 3
 - 2.2 Components required for operation 4
 - 2.3 Intended use of battery-powered clipping machine FAV5 CL 4
 - 2.4 Intended use of charger GT203/Li-ion battery GT201 4
 - 2.5 Operating principle of the battery-powered clipping machine FAV5 CL 4
 - 2.6 Operating principle of charger GT203/ Li-ion battery GT201 4
 - 2.6.1 Charging principle 4
 - 2.6.2 Charging time 4
- 3. Preparation and setup 5
- 4. Working with the battery-powered clipping machine FAV5 CL, and the charger GT203/ Li-ion battery GT201 5
 - 4.1 System set-up 5
 - 4.1.1 Connecting the accessories..... 5
 - 4.1.2 Attaching the blade..... 6
 - 4.1.3 Connecting the charger/Charging the Li-ion battery..... 6
 - 4.2 Functional check of the battery-powered clipping machine FAV5 CL 7
 - 4.3 Function checks of the charger 8
 - 4.4 Safe handling blade..... 8
 - 4.4.1 Sharpening the cutting plate 8
 - 4.4.2 Changing the blade 8
 - 4.4.3 Lubricating the blade..... 9
 - 4.4.4 Changing the Li-ion battery 9
- 5. Processing 10
 - 5.1 Cleaning/disinfection 10
 - 5.2 Control, care and inspection 10
- 6. Maintenance 11
- 7. Troubleshooting list 12
 - 7.1 Battery-powered clipping machine FAV5 CL .. 12
 - 7.2 Charger and Li-ion battery 12
- 8. Technical Service 14
- 9. Accessories/Spare parts 15
- 10. Technical data 15
 - 10.1 Battery-powered clipping machine FAV5 CL .. 15

10.2	Charger	15
10.3	Li-ion battery	16
10.4	Declaration of conformity	16
11.	Disposal	16
11.1	For EU countries only	16
11.2	Li-ion batteries	16
12.	Two year guarantee	17
13.	Distributor in the US/Contact in Canada for product information and complaints	17

- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- Keep the instructions for use accessible for the user.
- Always adhere to applicable standards.
- Ensure that the electrical facilities in the room in which the device is being used conform to IEC norms.
- Unplug the device by pulling the plug, but never by pulling the power cord.
- Do not operate the product in explosion-hazard areas.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.

1. Safe handling

CAUTION

Federal law restricts this device to sale by or on order of a physician!

Risk of fatal injury from electric shock!

- **Do not open the product (except for changing a charging bay).**
- **Connect the product only to a grounded power supply.**

- Prior to use, check that the product is in good working order.
- Store the charger and Li-ion batteries at room temperature.
- Keep dry the charger and Li-ion batteries.
- Charge the Li-ion batteries prior to first use.
- Observe our "Notes on Electromagnetic Compatibility (EMC)".
- To prevent damage caused by improper setup or operation, and in order not to compromise warranty and manufacturer liability:
 - Use the product only according to these instructions for use.
 - Follow the safety and maintenance instructions.
 - Ensure the manufacturer's approval before using accessories that are not mentioned in the instructions for use.

2. Product description

2.1 Scope of delivery

Designation	Art. no.
Battery-powered clipping machine FAV5 CL	GT300
Li-ion battery	GT201
Charger	GT203
Mains power cord (with device plug)	see Chapter Accessories/ Spare parts
Instructions for Use	TA013440

Note

The blade GT330 #10 (USA+UK) for battery-powered clipping machine FAV5 CL is included in the scope of delivery. Various blades are available for different applications!

Battery-powered clipping machine FAV5 CL

2.2 Components required for operation

- Battery-powered clipping machine FAV5 CL
- Blade
- Charger
- Mains power cord (with device plug)
- Li-ion battery

2.3 Intended use of battery-powered clipping machine FAV5 CL

The battery-powered clipping machine FAV5 CL is used for shearing small animals.

The battery powered clipping machine FAV5 CL is used with a blade to be selected according to the respective application.

2.4 Intended use of charger GT203/Li-ion battery GT201

The charger GT203 is used for charging the battery-powered clipping machine FAV5 CL and the Li-ion battery. There is one charging bay each for the two components.

2.5 Operating principle of the battery-powered clipping machine FAV5 CL

The battery-powered clipping machine FAV5 CL is switched on (off) by pushing the ON/OFF switch up (down).

2.6 Operating principle of charger GT203/Li-ion battery GT201

The charger GT203 is designed for the mains voltage/frequency range 100 V to 240 V / 50 Hz to 60 Hz.

To make the charger GT203 ready for operation, connect it to mains power by plugging in the mains plug.

The charger GT203 is equipped with two charging bays. The first charging bay is for charging the battery-powered clipping machine FAV5 CL, while in the second charging bay the Li-ion battery is charged.

Each charging bay has an indicator field on the charger front panel assigned to it.

Charging starts automatically, as soon as a Li-ion battery is inserted in the charging station.

The charging time in the charging bays depends on the charge state and capacity of the Li-ion batteries.

2.6.1 Charging principle

The Li-ion batteries are charged by constant current pulses.

The charge state of the Li-ion battery is monitored during charging. Monitoring of the charging curve ensures that the battery is fully charged without over-charging.

Additionally, the battery temperature and charging time are monitored as well.

The charger monitors the battery status and indicates the charge state via two LED indicators.

LED indicators:

- green = Battery fully charged
- red = Battery completely discharged

2.6.2 Charging time

Charging is stopped as soon as the maximum charging time is reached.

The charging time is approx. 50 min.

3. Preparation and setup

Non-compliance with the following instructions will preclude all responsibility and liability in this respect on the part of Aesculap.

- When setting up and operating the product, adhere to
 - national regulations for installation and operation,
 - national regulations on fire and explosion protection.
- operating advisories according to IEC-/VDE regulations.

DANGER

Fire and explosion hazard!

- **Do not operate the product in explosion-hazard areas.**

CAUTION

Malfunction of, or damage to the product caused by improper setup!

- **Do not expose the product to direct sunlight or moisture.**
- **Do not cover the ventilation slots of the product.**
- **Connect the product to the specified voltage of 100 V to 240 V/50 Hz to 60 Hz.**
- **Ensure that the mains power socket used for supplying the product is freely accessible.**

4. Working with the battery-powered clipping machine FAV5 CL, and the charger GT203/Li-ion battery GT201

4.1 System set-up

4.1.1 Connecting the accessories

Combinations of accessories that are not mentioned in the present instructions for use may only be employed if they are specifically intended for the respective application, and if they do not compromise the performance and safety characteristics of the products.

- Please contact your B. Braun/Aesculap Partner or Aesculap Technical Service with any inquiries in this respect, see Chapter Technical Service.

Battery-powered clipping machine FAV5 CL

4.1.2 Attaching the blade

Risk of injury caused by sharp cutting plates!

- Handle the blade of clipping machine FAV5 with appropriate caution.

Note

The blade GT330 #10 (USA+UK) for battery-powered clipping machine FAV5 CL is included in the scope of delivery. Various blades are available for different applications!

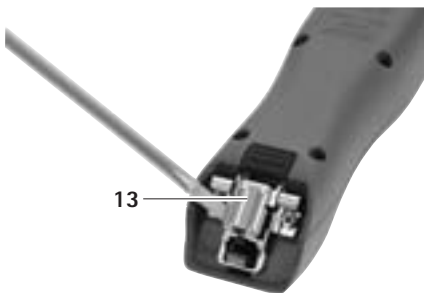
Fig. 1

- If the knife hinge is closed: Open knife hinge 13 with a screwdriver, see Fig. 1.

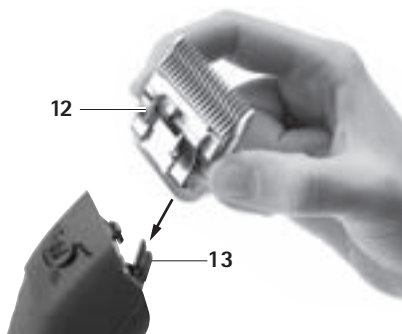
Fig. 2

- Install blade 12 on open knife hinge 13, see Fig. 2.
- Press ON/OFF switch 11.
- Press blade 12 gently in the direction of the clipping machine FAV5, see Fig. 2. The blade 12 engages with an audible click.
- Ensure that the blade 12 is fitted correctly on the clipping machine FAV5 CL.

4.1.3 Connecting the charger/Charging the Li-ion battery

Fire hazard due to battery poles shorted by fluids or metal components!

- Do not short the Li-ion battery.

Risk of injury and material damage due to damaged Li-ion batteries!

- Prior to charging, check the Li-ion batteries for any damage.
- Do not use or charge damaged Li-ion batteries.

Loss of capacity/performance due to prolonged storage of discharged Li-ion batteries!

- Store Li-ion batteries only in fully charged condition and recharge stored batteries once every month.

Destruction of the product, charging bay or Li-ion battery when inserting the wrong type of battery in the charging bay!

- Insert Li-ion batteries only in the appropriate charging bay.
- Insert the Li-ion battery in the correct position.

Note

Switch off the battery-powered clipping machine FAV5 CL before charging!

Note

Li-ion batteries heat up slightly during charging.

Note

Unplug the mains power cord at the mains socket when the charger is not going to be used!

- To connect charger **4**: Plug in the device plug of mains power cord **5** in charger **4**.
- Plug in the mains plug of mains power cord **5** at the mains power socket.
- Place Li-ion battery **8** in charging bay **6** of the charger **4** and charge.
 - or -
- Place battery-powered clipping machine FAV5 CL **2** in charging bay **10** of the charger **4** and charge.

Charge state indicator **7** or **9** lights up red.

After charging, the charge state indicator **7/9** shines green and the battery-powered clipping machine FAV5 CL **2**/Li-ion battery **8** is ready for operation.

4.2 Functional check of the battery-powered clipping machine FAV5 CL

- Carry out a visual inspection.
- Check the switch position.
- Install the battery, see Chapter Changing the Li-ion battery.
- Mount the blade correctly and lubricate it if necessary, see Chapter Safe handling blade head.
- Keep a charged Li-ion battery on hand, as reserve.
- Clean battery-powered clipping machine FAV5 CL prior to use, see Chapter Cleaning/disinfection.

Battery-powered clipping machine FAV5 CL

4.3 Function checks of the charger

- Carry out a visual inspection.
- Prior to connecting the product to mains power:
 - Check mains power cord **5** for possible damage.
 - Check the product for any damage (e.g. bent contacts of the charging bays).
- Plug in mains power cord **5** at the device socket of the charging station.
- Plug in the mains plug at the mains power socket.
- Place Li-ion battery **8** in charging bay **6** of the charger **4** and charge.
 - or -
- Place battery-powered clipping machine FAV5 CL **2** in charging bay **10** of the charger **4** and charge.

Charge state indicator **7** or **9** lights up red.

After charging, the charge state indicator **7/9** shines green and the battery-powered clipping machine FAV5 CL **2**/Li-ion battery **8** is ready for operation.

The product is ready for operation.

4.4 Safe handling blade

4.4.1 Sharpening the cutting plate

Note

For sharpening the cutting plate, please contact Technical Service, see Chapter Technical Service.

4.4.2 Changing the blade

Risk of injury caused by sharp cutting plates!

- Handle the blade of clipping machine FAV5 CL with appropriate caution.

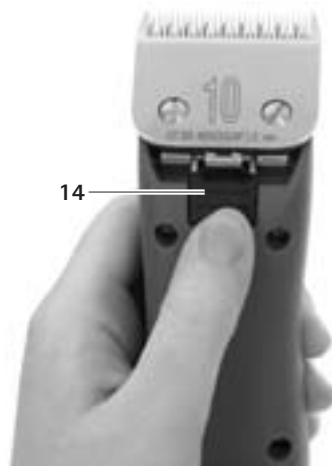
Fig. 3

- Press ON/OFF switch **11** to switch off battery-powered clipping machine FAV5 CL.
- Press blade lock **14**, see Fig. 3.

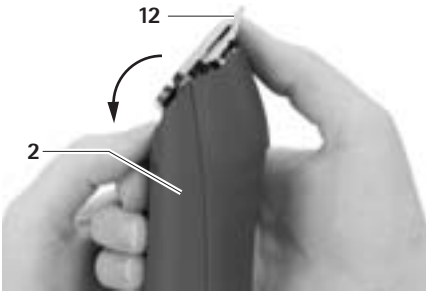
Fig. 4

- Fold forward blade **12**, see Fig. 4.
- Detach blade **12**.
- Clean open knife hinge with a brush.
- Install blade **12** on knife hinge.
- Plug in the power cord of battery-powered clipping machine FAV5 CL.
- Press ON/OFF switch **11**.
- Slightly push blade **12** towards battery-powered clipping machine FAV5 CL.
Blade **12** will click into position.
- Check that blade **12** is correctly seated on battery-powered clipping machine FAV5 CL.

4.4.3 Lubricating the blade

- Clean the upper and lower cutting plates with a soft cloth or brush. Make certain that the gaps between the cutting teeth are cleaned properly.
- Slightly lubricate the upper and lower cutting plates with special lubricant for blades GT604.

4.4.4 Changing the Li-ion battery

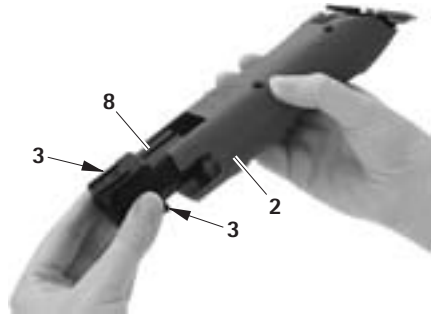
Fig. 5

- Hold battery-powered clipping machine FAV5 CL **2** and press battery release lock **3**.
- Pull out Li-ion battery **8** from battery-powered clipping machine FAV5 CL **2**.
- Insert the charged Li-ion battery **8** in battery-powered clipping machine FAV5 CL **2** as far as it will go.
The battery-powered clipping machine FAV5 CL **2** is ready for operation.

Battery-powered clipping machine FAV5 CL

5. Processing

CAUTION

Damage to, or destruction of the product caused by mechanical cleaning/disinfecting or sterilization!

- Only clean and disinfect the product manually.
- Do not sterilize the product under any circumstances.

CAUTION

Damage to the product due to incorrect cleaning or inappropriate cleaning/disinfecting agents!

- Use any commercial detergent for surface cleaning.
- Never immerse the product in water or detergent.
- Do not introduce any fluid into the charging bays.
- Clean the contacts in the charging bays, applying extreme care.

5.1 Cleaning/disinfection

Note

Always follow disposal and hygiene instructions and regulations!

Manual cleaning/disinfection

DANGER

Risk of electric shock and fire hazard!

- Unplug the device before cleaning.
- Ensure that no fluids will penetrate the product.
- Leave the product exposed to air for at least 1 minute after cleaning/disinfecting.

- Wipe the shell of the product with a lint-free cloth moistened with a commercial disinfectant for wipe disinfection.
- If necessary, after the appropriate exposure time, wipe off any residues of the cleaning/disinfecting agent with a lint-free cloth moistened with clean water.
- Dry the product with a clean, lint-free cloth.
- Clean the contacts in the charging bay with isopropanol or ethyl alcohol and a cotton swab. Do not use corrosive chemicals.
- If necessary, repeat the cleaning/disinfecting process.

5.2 Control, care and inspection

- Allow the product to cool down to room temperature.
- Inspect the product after each cleaning and disinfecting cycle to be sure it is: clean, functional, and undamaged.
- Lubricate blade if required, see Chapter Lubricating the blade head.
- Check the product for any damage, abnormal running noise, overheating or excessive vibration.
- Inspect the blade for broken, damaged or blunt cutting edges.
- Set aside the product if it is damaged.

6. Maintenance

To ensure reliable operation, Aesculap recommends servicing the product at least once a year.

For the respective services, please contact your national B. Braun/Aesculap agency, see Chapter Technical Service.

Service addresses

In the UK

KG PRODUCTS

243-251 City Road, Fenton

Stoke-on-Trent, Staffs

ST4 2PX

United Kingdom,

Phone (44) - 01782 844 866

DIAMOND EDGE

126 Gloucester Rd.

Brighton, BN1 4BU.

United Kingdom

Phone (44) - 01273 605 922

Or in the US

A & J Sales and Service

42 - A Frame Drive

West Burke, VT 05871

USA

Phone: (800) 447-1957

Or in Germany

Aesculap Suhl GmbH

Fröhliche-Mann-Straße 15

98528 Suhl / Germany

Phone: +49 (0) 3681 49820

Fax: +49 (0) 3681 498234

E-Mail: suhl@aesculap.de

Other service addresses can be obtained from the address indicated above.

Battery-powered clipping machine FAV5 CL

7. Troubleshooting list

7.1 Battery-powered clipping machine FAV5 CL

Malfunction	Detection	Cause	Remedy
Battery-powered clipping machine FAV5 CL does not start	Check switch position	Li-ion battery not loaded	Load Li-ion battery
	–	Li-ion battery defect	Insert new Li-ion battery
	–	Li-ion battery not inserted	Insert new Li-ion battery
Battery-powered clipping machine FAV5 CL has come to a stop	–	Li-ion battery deactivated through protective circuit	Deactivate and reactivate the control unit
	–	Li-ion battery is empty	Insert new Li-ion battery
Battery-powered clipping machine FAV5 CL does not load into the charging station	LED indicators do not light up	–	see Chapter Charger and Li-ion battery

7.2 Charger and Li-ion battery

Malfunction	Detection	Cause	Remedy
Charger not working	LED indicators do not light up	Mains power cord not plugged in	Plug mains power cord into the device socket on the charger and into the socket of the mains power supply.
		Li-ion battery not loaded	Replace mains power cord
	Charger defect	Send product to manufacturer for repair	

Malfunction	Detection	Cause	Remedy
Li-ion battery cannot be loaded	Li-ion battery in place, charge state indicator does not light up	Charger contacts dirty or damaged	Clean contacts in the charging bay, see Chapter Cleaning/disinfection or send to manufacturer for repair
		Li-ion battery defect	Send Li-ion battery to manufacturer for repair
		Charger defect	Send charger to manufacturer for repair
		Excess temperature is detected in the Li-ion battery during charging	Remove Li-ion battery from charging bay, leave to cool and start the charging process again. If the problem occurs again, send battery to manufacturer for repair
		Charging fault or Li-ion battery defect	Remove Li-ion battery from charging bay and repeat charging process, if necessary in another charging bay. If the problem occurs again send Li-ion battery to manufacturer for repair

Battery-powered clipping machine FAV5 CL

8. Technical Service

- For service, maintenance and repairs, please contact your national B. Braun/Aesculap agency.

Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

Service addresses

In the UK

KG PRODUCTS

243-251 City Road, Fenton

Stoke-on-Trent, Staffs

ST4 2PX

United Kingdom,

Phone (44) - 01782 844 866

DIAMOND EDGE

126 Gloucester Rd.

Brighton, BN1 4BU.

United Kingdom

Phone (44) - 01273 605 922

Or in the US

A & J Sales and Service

42 - A Frame Drive

West Burke, VT 05871

USA

Phone: (800) 447-1957

Or in Germany

Aesculap Suhl GmbH

Fröhliche-Mann-Straße 15

98528 Suhl / Germany

Phone: +49 (0) 3681 49820

Fax: +49 (0) 3681 498234

E-Mail: suhl@aesculap.de

Other service addresses can be obtained from the address indicated above.

9. Accessories/Spare parts

Art. no.	Design	Power cord	Li-ion battery	Charging station	Battery-powered clipping machine	Accessories
GT306	Europe outside of Great Britain	TA012170 Europe-flat plug, black, L=1.8 m	GT201	GT200880	GT300 unpacked	GT604800 Hand oiler
GT306G	Great Britain	TA012169 GB plug, black, L=1.8 m	GT201	GT200880	GT300 unpacked	GT604800 Hand oiler
GT306J	Japan, Mexico 110-125 V	TA012168 2-pin connector plug, black, L=1.8 m	GT201	GT200880	GT300 unpacked	GT604800 Hand oiler
GT306K	USA	TA012168 2-pin connector plug, black, L=1.8 m	GT201	GT200880	GT300 unpacked	GT604800 Hand oiler

10. Technical data

10.1 Battery-powered clipping machine FAV5 CL

Model	GT300
Stroke frequency	2 300/min
Charging voltage/Output voltage	max. 8.4 V
Charging current/Output current	max. 1.2 A
Weight (without blade and battery)	385 g
Weight (with blade and battery)	450 g
EN 60335-1, EN 60335-2-29	

10.2 Charger

Model	GT203
Mains voltage ranges (Current intake)	100-240 V
Frequency	50-60 Hz
Charging voltage/Output voltage	max. 8.4 V
Charging current/Output current	max. 1.2 A
Weight	495 g
Quality marks	CE, UL

Battery-powered clipping machine FAV5 CL

10.3 Li-ion battery

Model	GT201
Cell type	Li
DC voltage	7.4 V
Capacity	approx.1.2 Ah
Charging time	50 min ±5 min

10.4 Declaration of conformity

	<p>We declare under our sole responsibility that this product complies with the following standards and regulatory documents:</p> <p>2006/95/EC EC Low Voltage Directive DIN EN 60335-1 DIN EN 60335-2-8 DIN EN 60335-2-29 Safety of household and similar electrical appliances</p> <ul style="list-style-type: none"> - general requirements - for clipping machines - for battery chargers <p>Aesculap Suhl GmbH</p>
---	--

11. Disposal

Electrical tools, accessories and packaging should be sent for environmentally responsible recycling.

11.1 For EU countries only

	<p>Do not dispose of electrical tools in domestic waste!</p> <p>According to European Directive 2002/96/EG on old electrical and electronic appliances and its implementation in national law, electrical tools that are no longer suitable for use must be separately collected and sent for environmentally responsible recycling.</p>
---	--

11.2 Li-ion batteries

Do not dispose of batteries in domestic waste, fire or water. Defective or used batteries must be collected, recycled or disposed of in an environmentally-friendly way according to Directive 2006/66/EG.

12. Two year guarantee

Dear Customer,

Many thanks for choosing one of our products. The name Aesculap has a long-standing reputation for tried and tested quality and first-class service. Aesculap Suhl GmbH is certified according to DIN EN ISO 9001. We offer our customers innovative and high-performance products.

Aesculap manufactures high-quality devices and can assure you that it uses high-quality materials and careful manufacturing processes. If there is a quality defect in a product, we guarantee that product for two years after its purchase (starting from the invoice date).

In addition to your contractual and legal guarantee rights, Aesculap offers you a manufacturer's guarantee with the following conditions:

This manufacturer's guarantee is valid for two years starting from the date of purchase. It is valid for defects arising from material or manufacturing errors. The following are excluded from our manufacturer's guarantee:

defects caused by improper use, normal wear and tear, the erroneous action of unauthorized persons intervening with the product or unsuitable or unacceptable foreign products being used during the product's assembly. Defects that do not affect or only minimally affect the value or usability of the product are also excluded.

If a defect that falls under our manufacturer's guarantee occurs during our guarantee period of two years after the purchase date, we will choose either to repair the defective product or to replace it.

A prerequisite for our manufacturer's guarantee is that you send or hand in the product and purchase receipt (invoice) to one of our national Aesculap agencies or national responsible customer service centers within the applicable guarantee period.

The addresses of our customer service centers, see Technical Service.

Irrespective of our manufacturer's guarantee, you are also entitled to make a guarantee claim against the seller for a supplementary performance (rework, repair or exchange), cancellation or price reduction, insofar as this is within your legal or contractual rights. These rights are not affected by our additional manufacturer's guarantee.

13. Distributor in the US/Contact in Canada for product information and complaints

Aesculap Inc.
3773 Corporate Parkway
Center Valley, PA 18034
USA

Service addresses

A & J Sales and Service
42 - A Frame Drive
West Burke, VT 05871
USA
Phone: (800) 447-1957

Or in Germany
Aesculap Suhl GmbH
Fröhliche-Mann-Straße 15
98528 Suhl / Germany
Phone: +49 (0) 3681 49820
Fax: +49 (0) 3681 498234
E-Mail: suhl@aesculap.de

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